

# AccessIT 3 Telephone Access for Android

AccessIT 3 is a USB accessory that plugs into a Windows computer and allows users to control their Android phone via Snap Core First. It provides speakerphone option only.


## How to Setup AccessIT 3

- Insert the AccessIT 3 into the USB port (Note: AccessIT 3 requires USB 2.0 or higher)
- Enable Bluetooth - Windows Speech Devices
  - On the speech device, select the Start menu
  - Select Settings
  - Select Devices
  - Select Bluetooth
  - Move the Bluetooth dongle to the On position
- Enable Bluetooth - iOS (iPad) Speech Devices
  - On the speech device, go to Settings
  - Select Bluetooth and enable Bluetooth


## How do I access my phone from Snap Core First?

In Snap Core First

1. Go to **Dashboard**
2. Select **AccessIT – Android Phone** from the Dashboard

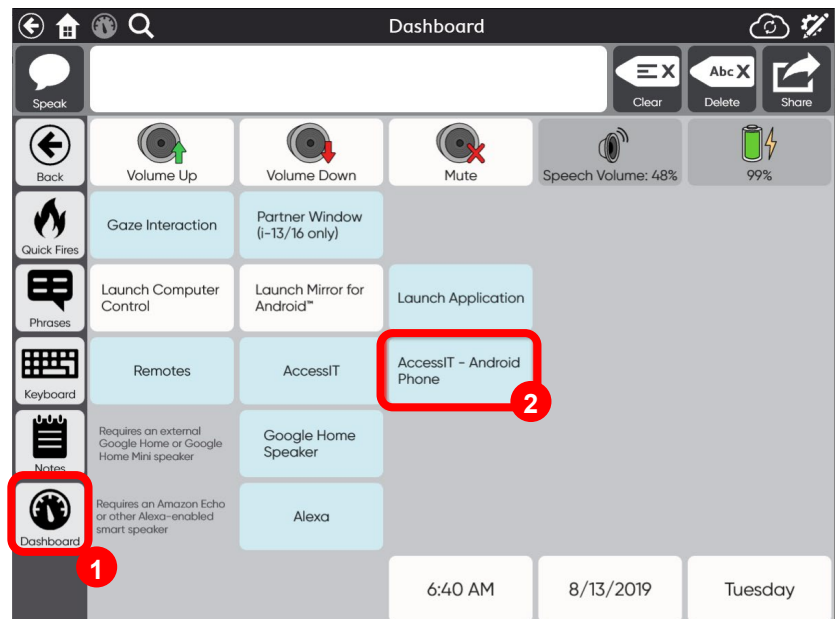

- Use the **arrow keys** to control your phone and navigate to the phone's calling app.
- Use the **Enter** key to open the app

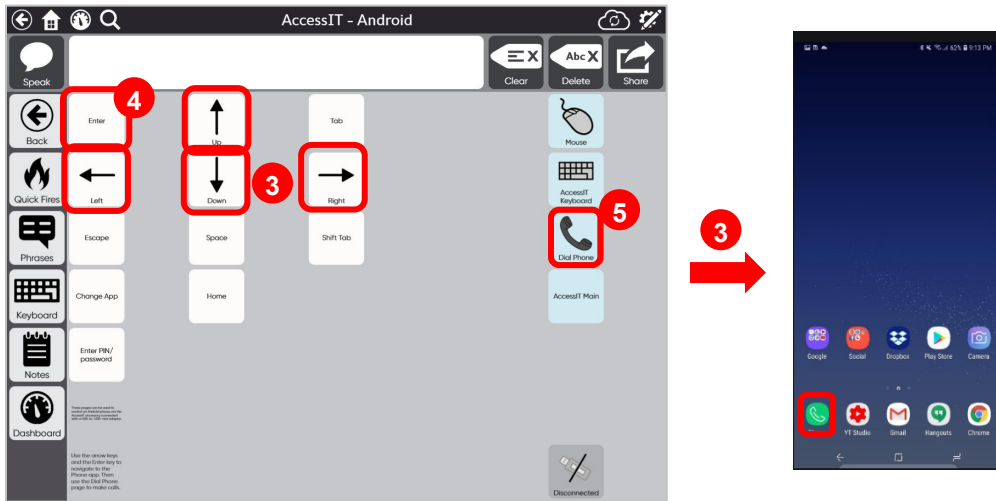

- The **Dial Phone** key in Snap Core First opens the phone call control screen for making phone calls

