




















Icons in Computer Control

This document provides a reference to the icons in Computer Control and their definitions.




Computer Control		
	Computer Control Logo	Launches the Computer Control application
Gaze Trace		
	Gaze Trace Locked Zone	<ul style="list-style-type: none"> Follows gaze, dwells and shrinks to confirm click Locks when a selection is made Unlocks when user exits the trace before dwell animation completes
	Activator Button	Works alongside the Gaze Trace Locked Zone and opens the Interaction Menu after a selection is made
Interaction Menu (at the center of the screen after activation)		
	Scroll Button	Scrolls the active window / screen up and down
	Right Click Button	Right-clicks the selection on the screen
	Keyboard Button	Brings up the keyboard on the screen
		On the keyboard view, swaps the keyboard to the top or bottom of the screen to accommodate the current cursor location
	Adjust Target Button	Corrects cursor placement to ensure more precise clicking on the specific screen selection
	Click & Drag Button	Gives freedom of clicking on the screen selection and holding/dragging it to any desired location (on the same screen or to a folder on the screen)
	Drop Button	Works with the Click and Drag Button to drop the selected item to the desired location after it's been dragged over
	Left Click Button	Left-clicks the selection on the screen
	Double Click Button	Double-clicks the selection on the screen
	Quick Start Guide Button	Appears on the left side of the screen when the Interaction Menu opens; shows a guide to the Interaction Menu

	Hide Guide Button	Closes the Interaction Menu Guide screen
---	-------------------	--




Off-Screen Menu (activated by gazing down out of the screen area)

	More Button	Opens the Off-Screen Menu <i>*The More Button appears when gazing down out of the screen area</i>
	Settings Button	Opens the Settings screen for specifying the activation, calibration, and keyboard options based on individual user's needs for personalized experience <i>**Refer to the "Settings in Computer Control" document for more information</i>
	Time Line Button	Shows the thumbnails of the applications that are currently active and open on the screen as well as the history of the earlier activities
	Continuous Click Button	Allows a continuous left click on the screen selection
	Pause Gaze Button	Pauses all of the current eye gaze activities; a yellow border around the screen will indicate gaze is paused
		The Start Gaze Button replaces the More Button on the off-screen Menu; it resumes the gaze activities
	Quick Calibrate Button	Opens the Calibration screen
	Sleep Button	Puts computer/device to sleep

Shortcut Panel (on the upper center of the Off-Screen Menu)

	Copy Button	Shortcut for copying the current selection on the screen to paste it on a new location
	Paste Button	Shortcut for pasting the last copied content on a desired location
	Cut Button	Shortcut for cutting the current selection on the screen to paste it on a new location

Back, Close and Exit

	Back Button	On the top right corner or the right side of the screen for returning to the previous screen
	Close Button	Closes the currently used function or Menu
	Exit App Button	Exits the Computer Control application; located on the top left corner of the Off-Screen Menu

The Computer Control software is under continuous improvement so there may be slight differences between the software and the training materials.