


Launch Applications or Files from Snap Core First




1. Select **Edit**




2. Select a button

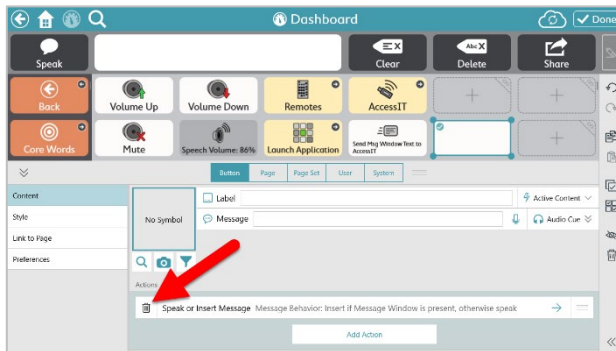


3. Optional – **Delete the current action to speak the label**

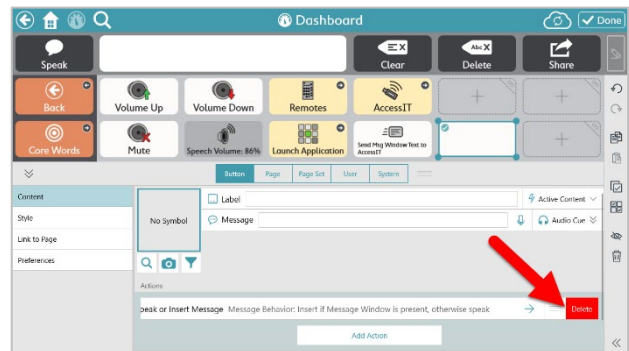


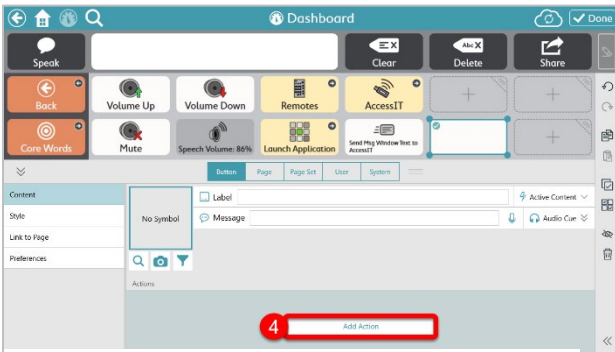


4. Select **Add Action**

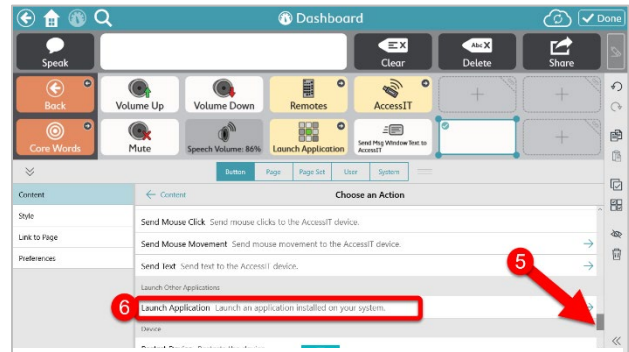


5. **Scroll down**
6. Select **Launch Other Application**

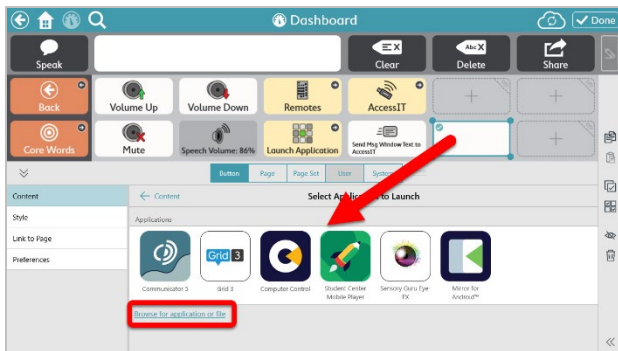


6. Select an **application shown** or
Select **Browse for application for file**

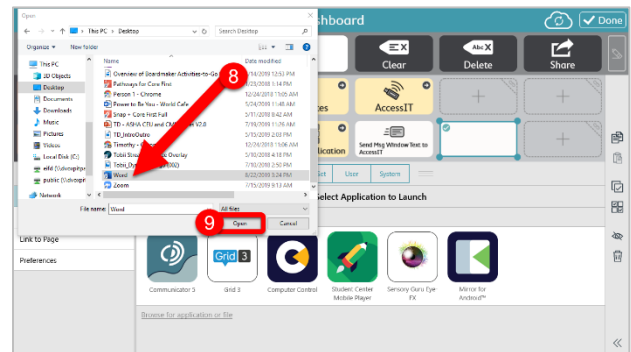

7. Locate and select the **application or file**
8. Select **Open**

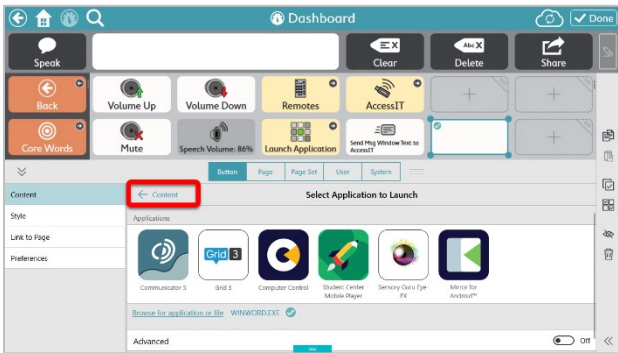

9. Select the **arrow** to return to the Content menu

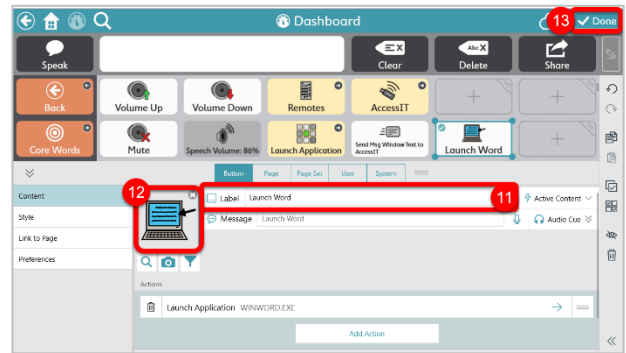

10. Update the button **Label**
 11. Update the button **symbol**
 12. Select **Done**



When this button is selected, your application or file will open!

REMEMBER: Depending on your user's access method, some applications may not be accessible without additional software support (i.e.: Windows Control or Computer Control for eye gaze users)