
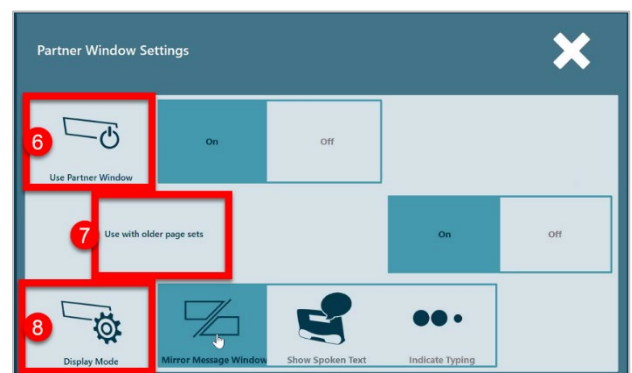


The Partner Window in Communicator 5

Partner Window Settings

1. Open **Communicator 5**
2. Open the **Quick Menu**
3. Select **Advanced Settings**
4. Select **Keyboards & Language**
5. Select **Partner Window**
6. Toggle On/Off **Use Partner Window**
7. Select if you want to **Use with older page sets**
8. Select your **Display Mode**

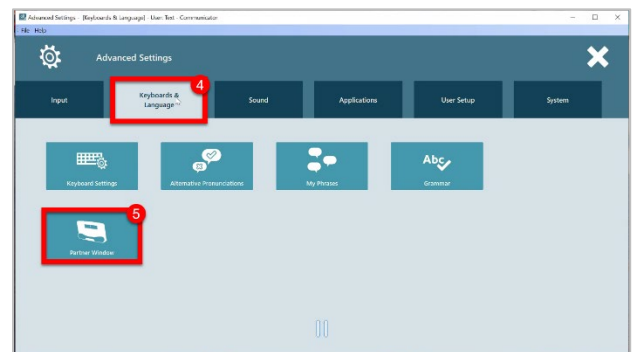

Note: When the Partner Window is turned on, an orange dot will appear in the upper right corner of the message window as an indicator to the AAC user




Display Modes



Mirror Message Window – as a message is being constructed, each letter and word will be displayed in the Partner Window as it is selected by the AAC user. The selected message will remain displayed on the Partner Window until the AAC user selects clear.




Show Spoken Text – as a message is being constructed, an ellipsis will be displayed in the Partner Window. Once the AAC selects speak, the complete message will be displayed in the Partner Window. This message will remain visible for a few moments then clear automatically from the Partner Window.

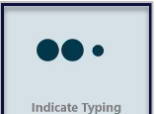


Indicate Typing – as a message is being constructed, an ellipsis will be displayed in the Partner Window. However, once the AAC selects speak, the complete message will **not** be displayed in the Partner Window.

Alternative Options to Access the Partner Window Settings

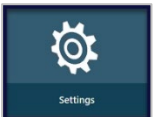


Use the **Settings Page Set** > Select **Page 2** to access the **Partner Window Settings**

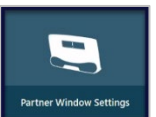

Add the **Partner Window Settings** page set directly to your homepage

1. Open the **Quick Menu**
2. Select **Edit Home Page**
3. Select **Add Page Sets**
4. Select **any communication level**
5. **Scroll down** on the left side menu
6. Select **Settings**
7. Select **Partner Window Settings**
8. Select **Add to Home Page**
9. **Exit** the Page Set Menu
10. **Exit** the Edit Home Page Menu

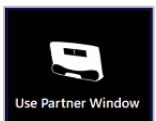

From your **Keyboard** select **Settings** > Select **Partner Window**