



# Re-sizable Buttons in Snap Core First

## Resizable Button Terminology

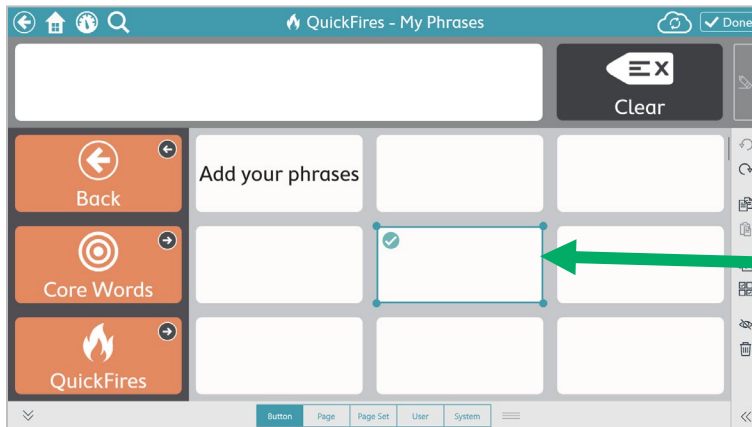

**Resize Frame:** the blue frame around a selected button in Edit mode

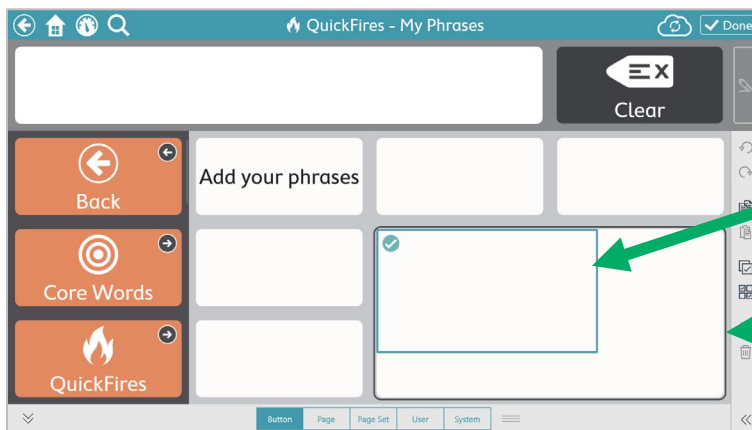

**Blue Frame:** indicates your touch

**Black Frame:** indicates what your resized button will look like if you release your touch

## Using the Re-sizable Button feature in Edit Mode

1. **Navigate** to the page you want to edit
2. Select **Edit**
3. Select a **button**
4. Select the **Re-size frame** around the button
5. Drag your mouse or touch to the desired button size indicated by the **Black Frame**
6. **Release** your click or touch