


Setting up the Text Page Set in Snap Core First


The Text Page Set

The Text Page Set can be accessed in two ways. For new users the Text Page Set can be accessed directly from the Set-up wizard, either at first launch of Snap Core First or when New User is selected from the Edit menu. For existing users, the Text Page Set can be accessed through the Edit Page Set menu.

Setting up the Text Page from the Set-up Wizard

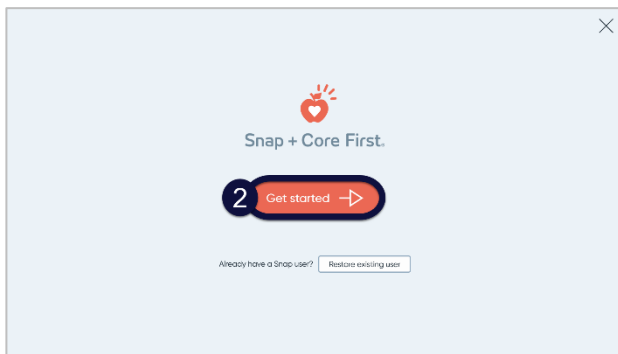


1. Launch the Set-up Wizard
2. Select **Get Started**

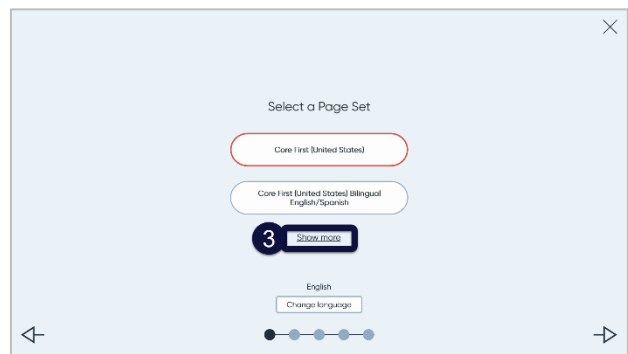


3. Select **Show More**




4. Scroll down
5. Select **Text Page Set (United States)**

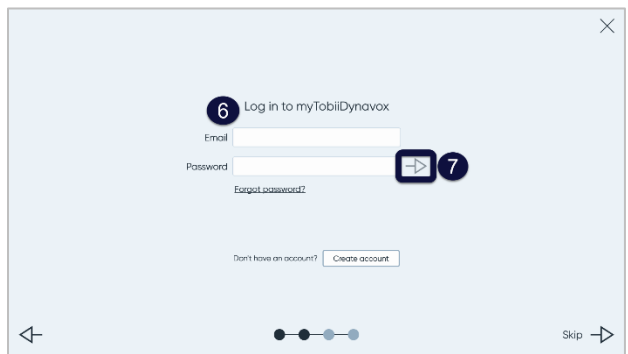


6. Log into your **mytobiidynamox** account
7. Select the **arrow** to continue

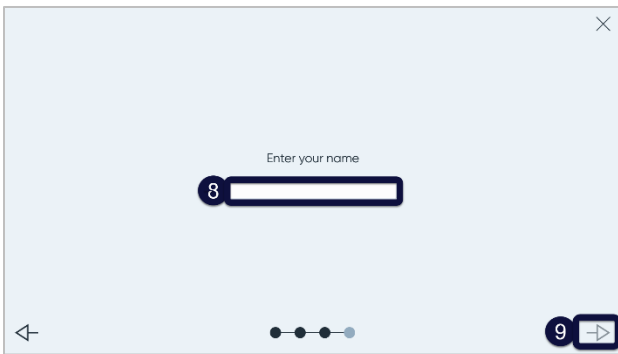


8. Enter the user's **name**
9. Select the **arrow** to continue

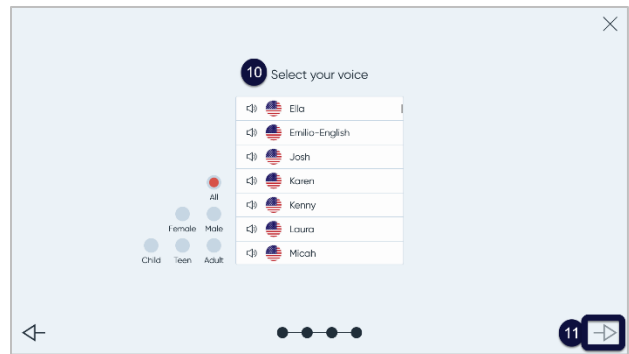

10. Select the user's **voice**
11. Select the **arrow** to continue

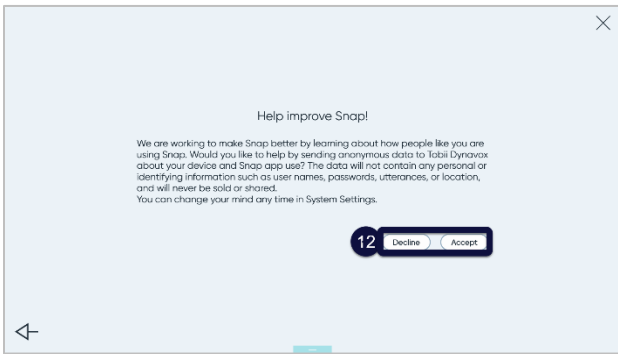


12. Select **Accept** or **Decline**

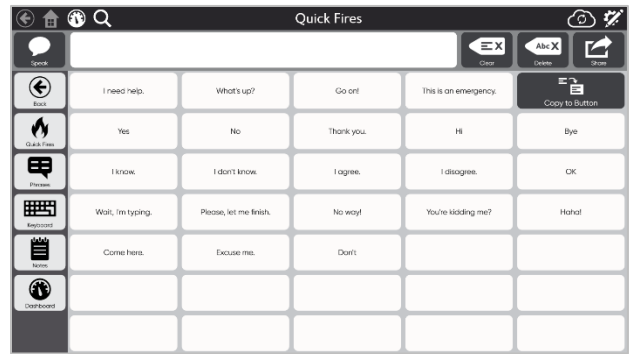


Start using the **Text Page Set!**


Adding the Text Page Set for an Existing User

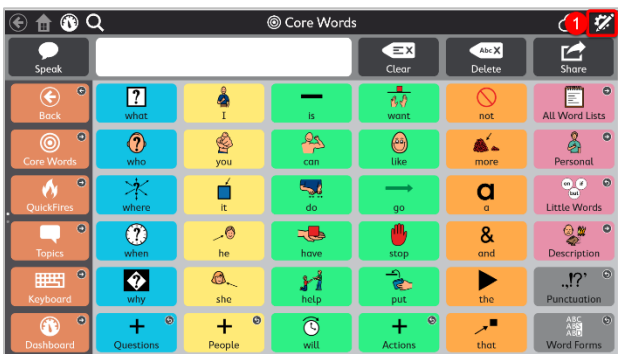


1. Select **Edit**

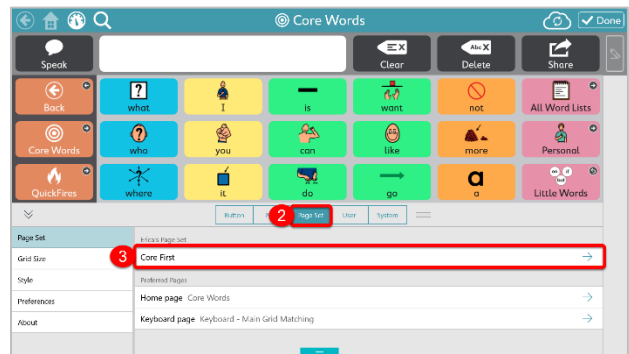

2. Select **Page Set**
3. Select the current **Page Set**

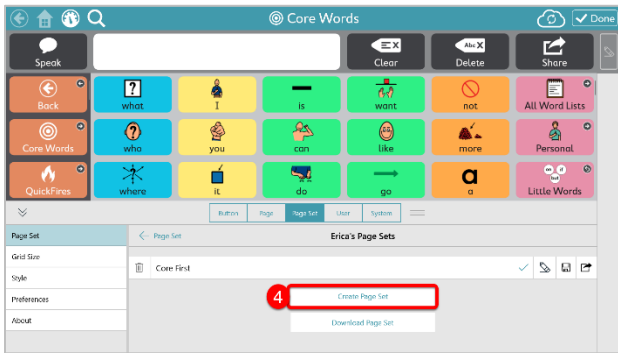


4. Select **Create Page Set**

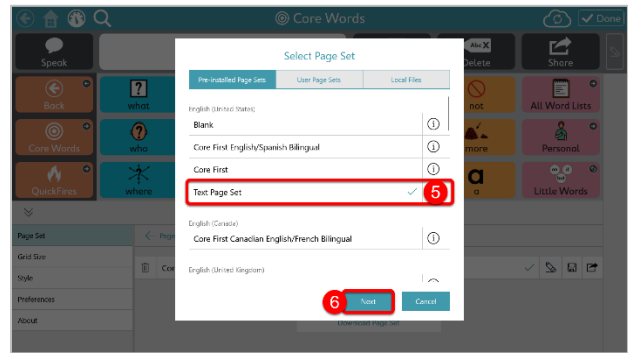


5. Select **Text Page Set**

6. Select **Next**

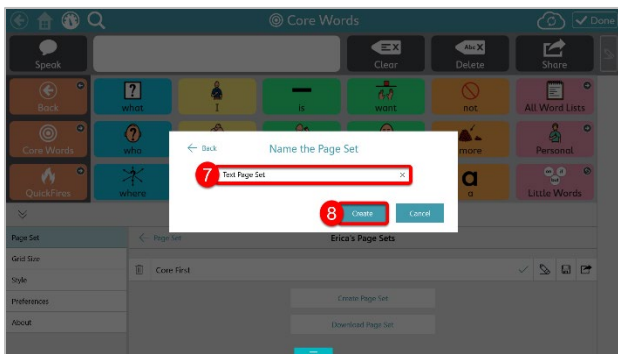

9. Name the Page Set if needed

10. Select **Create**

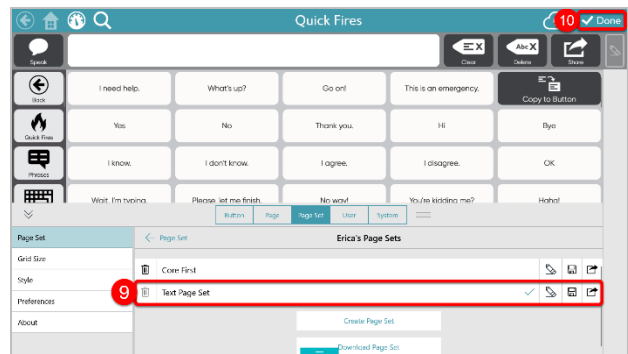

7. Be sure to select **Text Page Set**

8. Select **Done**



Start using the Text Page for communicating!