


Using the Partner Window in Snap Core First

Accessing the Partner Window Menu

1. Select **Edit**
2. Select **System**
3. Select **Device**
4. Select the options for your user

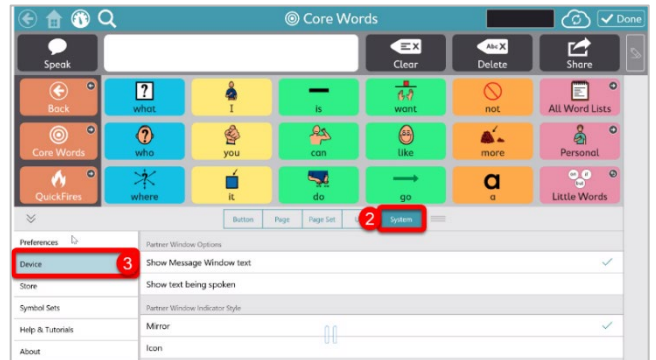



Partner Window Options

- **Show Message Window Text** - the Partner Window will display each letter or word as it's selected and sent to the message window. The message will continue to be displayed until the user selects Clear on their device.
- **Show Text being Spoken** – the Partner Window will display an ellipsis until the AAC user selects speak, then the complete message will display in the Partner Window.



Partner Window Indicator Style

- **Mirror** – when the Partner Window is turned on, the user will see a mirror image of the Partner Window on the top right side of the top menu bar. This will reflect exactly what the communication partner is viewing depending on the settings selected above.
- **Icon** – when the Partner Window is turned on, the user will see a solid icon on the top right side of the top menu bar,

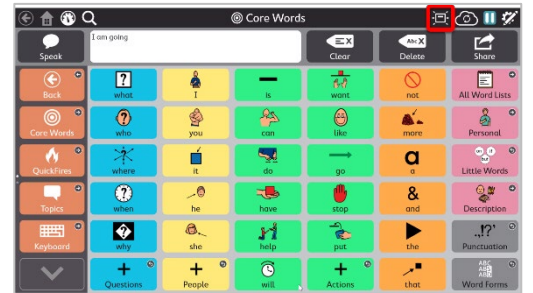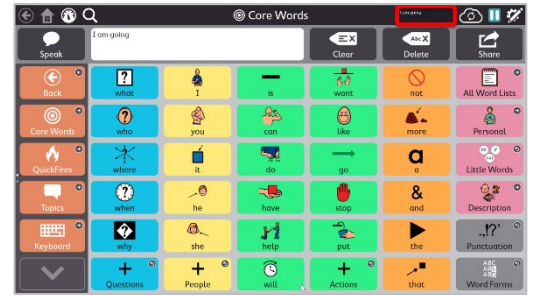

Using the Dashboard to Control the Partner Window

1. Select the **Dashboard** icon
2. Select the **Partner Window (I-13/16 only)** button
3. Use the **Turn Partner Window On/Off** to toggle the power of the Partner Window
4. Use the **Partner Window Brightness Up/Down** to adjust the brightness

