

Windows Access Icons in Communicator 5

Mouse emulation is the act of moving the mouse cursor around the screen following a user's eye gaze as if the user was moving a mouse or other pointing device. Windows Access in Communicator 5 mimics mouse emulation and allows for interaction with sensory games such as Eye FX and Look to Learn. This document provides a reference for all icons associated to Windows Access in Communicator 5.



Locate the Windows Access Page Sets in Communicator 5

In Communicator 5, Windows Access page sets are in the **Text Communication > Windows Access** category.



Placement of Windows Access Toolbars


Three options are available for where to place the Windows Access toolbars:

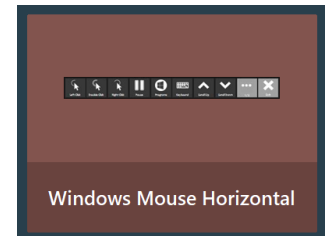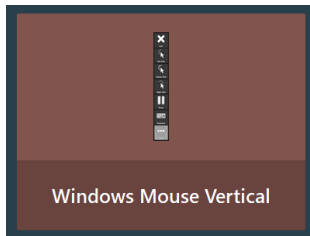

Windows Mouse Vertical – The toolbar shows on the left (default) or the right side of the screen.


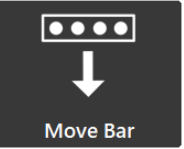
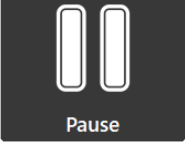
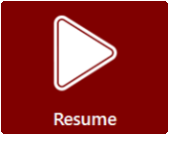
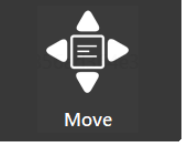
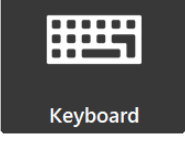
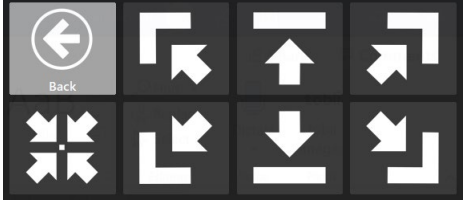
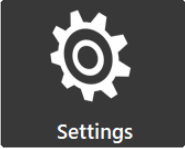

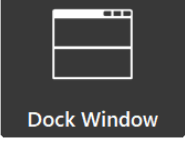




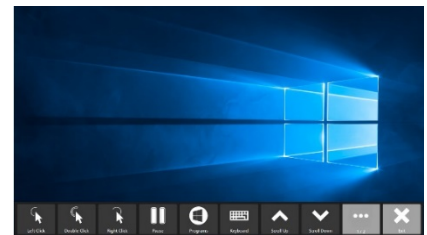


Windows Mouse Floating – The toolbar shows on one of the 7 areas specified by the user: center, top left, top center (default), top right, bottom left, bottom center, and bottom right.

Windows Mouse Horizontal – The toolbar shows on the bottom (default) or the top of the screen.



Windows Access Icons

<p>Exit</p>	<p>Closes and exits Windows Access toolbar and returns to Communicator 5 Home Page</p>	<p>Move Bar</p>	<p>Moves the vertical toolbar from the left side of the screen to the right side</p>
<p>Left Click</p>	<p>Left clicks the selection on the screen</p>	<p>Move Bar</p>	<p>Moves the vertical toolbar from the right side of the screen to the left</p>
<p>Double Click</p>	<p>Double clicks the selection on the screen</p>	<p>Move Bar</p>	<p>Moves the horizontal toolbar from the bottom of the screen to the top</p>

 <p>Right Clicks the selection on the screen</p>	 <p>Moves the horizontal toolbar from the top of the screen to the bottom</p>
 <p>Pauses the current gaze activity; switches to the Resume button</p> 	 <p>Moves the floating toolbar to a desired screen location among one of the 7 choices:</p>
 <p>Displays the keyboard on the screen</p>	<p>Top Top Top</p> <p>Left Center Right</p>  <p>Center Bottom Bottom Bottom</p> <p>Left Center Right</p>
 <p>Opens the Windows Access Settings screen (*Refer to the Communicator 5 Windows Access Settings document for details)</p>	 <p>Indicates currently on the page 1 of the toolbar and switches to the page 2 of the toolbar</p>
 <p>Displays the icons of the programs configured in the Program Shortcut Settings (e.g., the Google Chrome icon) for easy access to frequently used programs</p>	 <p>Indicates currently on the page 2 of the toolbar and switches to the page 1 of the toolbar</p>
 <p>Adjusts active application window to the edge of the Windows Access toolbar for a better view of both windows</p>	 <p>Scrolls up the active window if applicable</p>
 <p>Scrolls down the active window if applicable</p>	 <p>Returns to the previous toolbar screen</p>
 <p>Confirms the successful launch of a selected program in the Programs list and returns to the main toolbar</p>	 <p>Confirms the successful launch of a selected program in the Programs list and returns to the main toolbar</p>